


Menicon Z Progressive Near

Design: Central near vision progressive

Simultaneous vision

Front surface: Z Progressive: Progressive aspheric design with near vision at the centre.

Back surface: Sphero aspheric design.

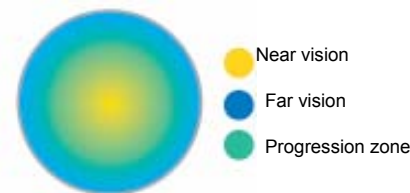

Parameter range

Menicon Z Progressive Near

Parameters*	r_0 (mm)	Power (D)	Diameter (mm)	Addition (D)
Min / Max	6.50 to 9.00	- 25.00 to + 25.00	9.10 10.10	1.00 to 3.00
Standard	7.20 to 8.40	+ 5.00 to - 10.00	9.60	
Steps	0.05	0.25	*	0.50

Indications

- Presbyopia correction : to improve Near vision
- Menicon Z Progressive Near on the non dominant eye combined with a well centred Menicon Progressive lens on dominant eye

Modified binocular vision

Menicon Z Progressive on dominant eye / Menicon Z Progressive Near on non dominant eye



- **Recommended lens replacement < 2 years. Daily wear.** C.A. = Corneal Astigmatism I.A. = Internal Astigmatism

Fitting rules

Menicon Z Progressive Near

\varnothing_T	$K - K' < 0.20$ mm	0.20 mm $\leq K - K' \leq 0.40$ mm
9.10 9.60 10.10	$r_0 = K + 0.05$	$r_0 = K$

K = flattest corneal radius K' = steepest corneal radius

- Sphere = lens refraction sphere (in minus cylinder) ± 0.25 D
- Addition = Spectacles addition (do not over-correct).
- Observations: Good centration in DV

Lens care

MeniCare Plus: for daily care.

Progent: once a week for intensive cleaning (protein removal) and disinfection.

