


Menicon Z

Design: Front and back aspheric

Menicon Z Material

Dk / t_c: 125 x 10⁻⁹ (ISO) (t_c: 0.13 mm for -3.00 D)

Diameter	9.00	9.20	9.40	9.60	9.80
r ₀ in 0.10 mm steps					
7.20					
7.30 ~ 7.50					
7.60 ~ 7.80					
7.90 ~ 8.10					
8.20 ~ 8.60					

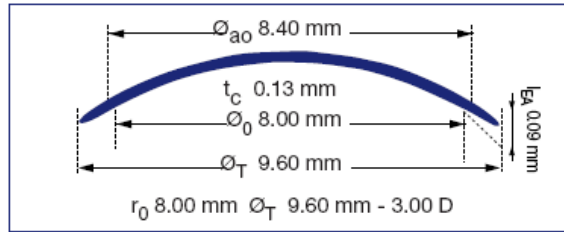

Parameter range

Parameters*	r ₀ (mm)	Spherical power (D)	Diameter (mm)
Min / Max	6.50 to 9.00	- 25.00 to + 25.00	Free choice of diameter from 9.00 to 9.80
Standard	7.20 to 8.40	+ 5.00 to - 10.00	Diameter depending on r ₀
Steps	0.10	0.25	0.20

* Other parameters may be available upon request.

Fitting rules

Ø _T	K-K' < 0.20 mm	0.20 mm ≤ K-K' ≤ 0.40 mm	0.40 mm ≤ K-K' ≤ 0.50 mm
Diameter 9.00 to 9.80 depending on radius	r ₀ = K + 0.10	r ₀ = K	r ₀ = K - 0.10
	K flattest corneal radius	K' steepest corneal radius	r ₀ lens base curve radius

Indications

- Hyper DK RGP lenses for myopia, hypermetropia and/or corneal astigmatism correction.
- Recommended for:
 - Kids and teens
 - Soft contact lens intolerance
 - High ametropias
 - Irregular corneas
 - Tear problems
 - Corneal astigmatism
 - Continuous wear (up to 30 days)
- Recommended lens replacement: < 2 years.
- Daily wear up to 30 days continuous wear.

Lens care

- **MeniCare Plus:** for daily care.
- **Progent:** once a week for intensive cleaning (protein remover) and disinfection.

