

2 week replacement
Silicone Hydrogel Contact lenses

2 week replacement
Silicone Hydrogel Contact lenses

Menicon PremiO Menicon PremiO Toric

Menicon PremiO Range

Recommended wearing schedule

Daily wear or flexible wear (up to 6 night extended wear)
2 week replacement.

Lens Care

MeniCare Soft

Multipurpose solution for cleaning, disinfection, storage and rinsing.

Innovative formulation : Comfortec™

- Efficient to remove lipids deposits.
- Less protein adhesion.
- High disinfection power with low cytotoxicity.



PremiO Club

All about Menicon premiO lenses,
e-memo service...



www.meniconpremio.com


CE 0483 manufacturer : Menicon Co.,Ltd.



Material

MeniSilk™ & Nanogloss™ technology

- Asmofilcon A.
- Silicone Hydrogel (Groupe V ISO18369-1).
- Water content : 40%.
- **Dk : 129×10^{-11}** (cm²/sec) (mLO₂/(mL · mmHg)) (ISO).
- Spectral and luminous transmittance : 98%.
- Refractive index : 1.423.
- Contact angle : 27° (Captive bubble method).
- Blue handling tint.
- Nanogloss™ surface treatment.

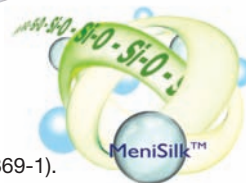

ME074/2 2011-01



Menicon PremiO

2 week replacement



Lens design

- $Dk/t_c: 161 \times 10^{-9}$ (cm/sec) (mL₀2 / (mL-mmHg)) ($t_c: 0.08$ mm @ -3.00 D).
- **Maximum corneal oxygenation across all parts of the lens and over the entire power range.**
- Sphero-aspheric design.
- Comfortable thin design with semi-round edges.
- Lens mark with the number 6 or 3 to read inside the lens.



Range

BC : 8.60 and 8.30 mm
 \varnothing_T : 14.00 mm
 F'_V : + 6.00 to - 13.00 D



Fitting

BC : 8.60 mm as first choice.

Packaging

- **6 lens blisters per pack**
Menicon heart shaped blister.
- **6 month PremiO Pack :**
4 boxes x 6 lenses
+ 3 x 360 ml de MeniCare™ Soft
+ 3 lense cases.



Menicon PremiO Toric


for astigmatism



Lens design

Back toric **Visiostable Design™**

- Asymmetric double slab-off design horizontal right and left ballast (without optical prism)
- Highest transmissibility among soft toric lenses
 $Dk/t_c: 161 \times 10^{-9}$ ($t_c: 0.08$ mm@-3.00)
- Double guide mark (in the 6 o'clock position)
- The lens should be placed on the cornea with the number 6 on top and the guide mark pointing down at 6 o'clock.

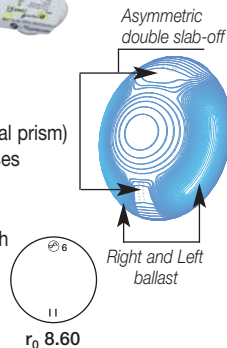


Range

BC : 8.60 mm
 \varnothing_T : 14.00 mm
 F'_V : plan to - 10.00



Cyl	-0,75	-1,25	-1,75
Axis	180° 170° 160° 90° 20° 10°	180° 170° 160° 90° 20° 10°	180° 170° 160° 20° 10°

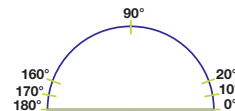

Fitting criteria

- Check good lens centration and stability.
- Axis stabilization (slit lamp) :
Guide mark at 6 o'clock.
Rotation should not be accepted over $\pm 15^\circ$.
- Check visual acuity and spherical over-refraction.

Packaging

- **6 lens blisters per pack**
- **PremiO Toric 6 month Pack :**
4 menicon PremiO Toric boxes
+ 3 x 360 ml de MeniCare™ Soft
+ 3 lense cases.



