CO D. 227050 Rev. 02 Data 2025/01 LINEA Soleko

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FUSTELLA 110x176 mm COLOR CMYK

## I-FRESH COMFORT EYE DROPS

SOLEKO Menicon GROUP

EN

**PROPERTIES AND INDICATIONS:** I-Fresh Comfort eye drops, preservative-free, is a lubricating and refreshing solution designed to relieve and help alleviate the symptoms of dry eyes caused by environmental factors such as sun, wind, air conditioning, or prolonged contact lens use. It is suitable for use with soft and rigid gas permeable contact lenses. I-Fresh Comfort contains hyaluronic acid along with a balanced mixture of fluid extracts from Aloe, Hamamelis, Centaurea Cyanus, Chamomile and Euphrasia. Hyaluronic acid, a molecule with strong hygroscopic and viscoelastic properties, stabilises the tear film and reduces the dehydration of ocular mucosa. The patented self-preserving dispensing system ensures no risk of contamination after opening, perfect dispensing performance with no residual drop and optimal dosing control.

**INSTRUCTIONS FOR USE:** Wash, rinse and dry your hands. Shake the bottle before use. Instill one or two drops into the eye as needed and blink 2 to 3 times. If mild discomfort occurs while wearing contact lenses, instill 1 or 2 drops and blink several times. If discomfort persists, remove the contact lens and contact your Eye Care Professional for a thorough examination.

**PRECAUTIONS:** Do not use the product if the seal has been tampered with or if the bottle is damaged. Close the bottle tightly after use. Use within 90 days of first opening and do not use the product beyond the expiry date. Adhere to the solutions recommended by your Eye Care Professional. For external ophthalmic use only. Seek medical advice if you are currently taking other medications. Store at a temperature between 5°C and 30°C.

**WARNINGS:** The bottle nozzle should not come into contact with the eye during application. Keep out of reach of children. DO NOT SWALLOW.

**CONTRAINDICATIONS:** Do not use the product if you are allergic or sensitive to any of the ingredients.

**CONTENTS:** Aloe 0.8%, Hamamelis 0.2%, Centaurea cyanus 0.2%, Chamomile 0.2%, Euphrasia 0.2%, Hyaluronic Acid 0.15%, Polyvinyl Alcohol 0.05%, Sodium Chloride, Bibasic Sodium Phosphate, Monobasic Sodium Phosphate, Purified water.

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Menicon Swiss GmbH, Dachacherstrasse 27, 4600 Olten, Switzerland