CO D. 227047

Rev. 02

Data 2025/01

LINEA Soleko

Prodotto GYY

Tipo Leaflet

FUSTELLA 110x176 mm

COLOR CMYK

I-FRESH MOISTURE EYE DROPS

SOLEKO Menicon

EN

PROPERTIES AND INDICATIONS: I-Fresh Moisture is ideal for relieving irritation and restoring the hydration of the ocular surface caused by dust, pollen, and environmental factors. It is also suitable for use with soft and rigid gas permeable contact lenses. I-Fresh Moisture contains 0.4% Hyaluronic Acid, a moisture-enhancing ingredient. Hydroxyethylcellulose in eye drops enhances tear film performance for hydrogel contact lens wearers. I-Fresh Moisture has a balanced formula with the pH of human tears. I-Fresh Moisture eye drops are gentle on the eye and suitable for regular use.

INSTRUCTIONS FOR USE: Wash, rinse and dry your hands. Shake the bottle before use. Instill one or two drops into the eye as needed and blink 2 to 3 times. If mild discomfort occurs while wearing contact lenses, instill 1 or 2 drops and blink several times. If discomfort persists, remove the contact lens and contact your Eye Care Professional for a thorough examination. I-Fresh Moisture can be used as required throughout the day, always adhering to the instructions provided by your Eye Care Professional.

PRECAUTIONS: Do not use the product if the seal has been tampered with or if the bottle is damaged. Close the bottle tightly after use. Use within 90 days of the first opening and do not use the product after the expiry date. Adhere to the solutions recommended by your Eye Care Professional. For external ophthalmic use only. Seek medical advice if you are currently taking other medications. Store at a temperature between 5°C and 30°C.

WARNINGS: The bottle nozzle should not come into contact with the eye during application. Keep out of reach of children. DO NOT SWALLOW.

CONTRAINDICATIONS: Do not use the product if you are allergic or sensitive to any of the ingredients.

CONTENTS: Polyhexamethylenebiguanide 0.0002%, Hydroxyethylcellulose 0.2%, Hyaluronic acid 0.4%, Sodium chloride, Dibasic sodium phosphate, Monobasic sodium phosphate, Purified water.

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Menicon Swiss GmbH, Dornacherstrasse 27,