CO D. 227030 Rev. 01 Data 2024/06 LINEA Soleko

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Tipo Leaflet

FUSTELLA 110x176 mm

COLOR CMYK

I-FRESH YAL EYE DROPS

EN

PROPERTIES AND INDICATIONS: I-Fresh Yal is ideal for restoring moisture both before and during use of any type of contact lens, especially in cases of dry and tired eyes with decreased tear production. I-Fresh Yal is also helpful in relieving eye irritation caused by dust, pollen, air pollution and other environmental factors. I-Fresh Yal has a balanced formula with the pH of human tears. Suitable for use with soft and RGP contact lenses. Suitable also for contact lens wearers. I-Fresh Yal contains Hyaluronic Acid, a moisture enhancer. Hydroxyethylcellulose in eye drops improves tear film performance in hydrogel contact lens wearers. I-Fresh Yal eye drops are gentle on the eye and suitable for regular use.

INSTRUCTIONS FOR USE: Wash your hands thoroughly. Shake the bottle before use. Administer one or two drops into the eye as needed and blink 2/3 times. If mild discomfort is encountered while wearing contact lenses, administer 1 or 2 drops and blink several times. If discomfort persists, remove the contact lens and contact your specialist for a thorough examination. I-Fresh Yal can be used as required throughout the day, always adhering to the instructions provided by your specialist.

PRECAUTIONS: For external ophthalmic use only. Do not use if the seal shows signs of tampering or if the bottle is damaged. Close the bottle well after use. Use within 30 days from the first opening and refrain from using the product beyond the expiry date.

Seek medical advice if you are currently taking other medications.

Store at a temperature between 5 and 30 °C.

WARNINGS: The bottle nozzle should not come into contact with the eye during application. Keep out of reach of children. DO NOT SWALLOW.

CONTRAINDICATIONS: Do not use the product if you are allergic or sensitive to any of the ingredients.

CONTENTS: EDTA 0.023%, PolyHexaMethyleneBiguanide 0.00005%, Hydroxyethylcellulose, Hyaluronic Acid 0.2%, Sodium Chloride, Dibasic Sodium Phosphate, Monobasic Sodium Phosphate, Anhydrous sodium hydroxide, Purified water.

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Menicon Swiss GmbH, Dornacherstrasse 27, 4600 Olten, Switzerland