## MENICARE PURE

MULTIPURPOSE SOLUTION

EN

**Instructions for use for *MeniCare Pure***

Multipurpose Solution for all rigid gas permeable contact lenses

This Instructions For Use contains important information.

Please read carefully and retain for future use.

INDICATIONS
<ul style="list-style-type: none"><li>DISINFECTING: Effective after 30minutes</li> <li>CLEANING</li> <li>RINSING</li> <li>STORAGE</li></ul>
TO OPEN

The dropper lip is sealed. To open, screw the cap clockwise as far as it will go. The spike inside the cap will pierce the seal.

DIRECTIONS
<p>Wash your hands thoroughly before handling your contact lenses:</p> <ul style="list-style-type: none"><li>Rub the lens carefully with several drops of <i>MeniCare Pure</i> for 20 seconds.</li> <li>Place the lenses in the right and left lens holder and rinse them with <i>MeniCare Pure</i>.</li> <li>Fill the lens case with <i>MeniCare Pure</i> and soak the lenses for at least 30 minutes or overnight.</li> <li>Before insertion, rinse the lenses with <i>MeniCare Pure</i>.</li> <li>Keep the lens case clean and dry after use.</li></ul>

Lenses may be stored in the unopened case until ready to wear, up to a maximum of 30 days. If you store your lenses for longer periods of time, they must be cleaned and disinfected with *MeniCare Pure* solution every 30 days.

WARNINGS
<ul style="list-style-type: none"><li>Never use tap water or non-sterile water with contact lenses as microbial contamination can occur, which may lead to permanent eye damage.</li> <li>Consult your doctor or eye care professional before using any eye medication with contact lenses.</li></ul>
CONTRA-INDICATIONS
<ul style="list-style-type: none"><li>Do not use in case you are allergic to one of the ingredients.</li></ul>
PACKAGING

*MeniCare Pure* is available in 250ml, and 70ml, bottles, including a disposable lens case.

PRECAUTIONS
<ul style="list-style-type: none"><li>Once opened, use <i>MeniCare Pure</i> within 3 months and make sure that the bottle is tightly closed after each use.</li> <li>Do not touch the bottle top with your hands.</li> <li>Do not swallow.</li> <li>Keep out of reach of children.</li> <li>Do not use after expiry date.</li> <li>Do not use if the film around the cap is broken before first use.</li> <li>Do not use with soft contact lenses.</li> <li>Keep the lens case clean and dry after use.</li> <li>Store at room temperature. (±25°C) <span><span>↕</span><sup>m</sup></span></li> <li>Store out of sunlight. <span><span>☀</span></span></li> <li>Replace the lens case with every new bottle of <i>MeniCare Pure</i>.</li> <li>Consult your eye care professional before changing solutions to a different solution.</li></ul>
ADVERSE REACTIONS

If you experience persistent discomfort or irritation, remove your lenses, stop using the product and consult with your eye care professional. If any serious incidents to the care products occur, report it to the authorized representative and the competent authority.

CONTENTS
<ul style="list-style-type: none"><li>E-Polylysine 0,01% as preservative,Edetate disodium 0,05% as chelating agent, HCO-60 0,1% as surfactant.</li></ul>
COMPOSITION

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© 2022, Menicon Co., Ltd.