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© 2024, Menicon Co., Ltd.

This instructions for use contains important information.

Please read carefully and retain for future reference.

# MENICARE SOFT

**1P MULTIPURPOSE SOLUTION** for all soft contact lenses

MeniCare Soft is indicated for disinfection, cleaning, protein removal, rinsing, storage of all soft contact lenses.

**TO OPEN (for 70mL)**

The dropper tip is sealed. To open, screw the cap clockwise as far as it will go. The spike inside the cap will pierce the seal.

**DIRECTIONS**

- Always wash and rinse your hands before handling your contact lenses.
- Always handle the same lens, right or left, first in order to avoid confusion.
- After removing the lens from your eye, place the lens in the palm of your hand.
- Apply a few drops of MeniCare Soft solution to the lens. Rub the lens gently for 20 seconds using your forefinger.
- Rinse the lens completely free from debris with fresh MeniCare Soft solution.
- Place the lenses in the right and the left lens case filled with MeniCare Soft solution and soak them at least 4 hours for disinfection (or overnight).
- Your lenses are now clean and disinfected.
- Before wear, rinse your lenses with MeniCare Soft solution.
- Pour the MeniCare Soft solution out of the lens case and rinse the case with fresh MeniCare Soft solution. Leave the case with the caps off to dry in the air. Place back the caps. Your case is now ready for use again.

Lenses may be stored in the unopened case until ready to wear, up to a maximum of 30 days. If you store your lenses for longer periods of time, they must be cleaned and disinfected with fresh MeniCare Soft solution every 30 days.

WARNINGS
<ul style="list-style-type: none"><li>Always use fresh MeniCare Soft solution – never re-use or “top-off” old solution in your lens case since solution reuse reduces effective lens disinfection and could lead to serious eye infection.</li> <li>Never use tap water or non-sterile water with contact lenses as microbial contamination can occur, which may lead to permanent eye damage.</li> <li>Keep out of reach of children.</li> <li>Consult your doctor or eye care professional before using any eye medication with contact lenses.</li></ul>
PRECAUTIONS
<ul style="list-style-type: none"><li>Do not use after expiry date.</li> <li>Do not use if the film around the cap is broken or the tamper evident ring is damaged before first use.</li> <li>Once bottle is open, discard any remaining solution after 3 months.</li> <li>Close bottle after use.</li> <li>Do not touch the bottle top with fingers.</li> <li>Store below 25°C.</li> <li>Store out of direct sun light.</li> <li>Do not swallow.</li> <li>Keep the lens case clean and dry; renew if regularly.</li> <li>Consult your eye care practitioner before changing solutions.</li> <li>Do not use for Rigid Gas Permeable contact lenses.</li></ul>
CONTRAINDICATIONS
<p>People allergic to any ingredient with MeniCare Soft Solution should not use this product. Never use MeniCare Soft Solution for heat disinfection.</p>
ADVERSE REACTIONS
<p>The following problems may occur: eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on the eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eye, unusual eye secretions, redness of the eye, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), or dry eyes. If you experience any of the above in the eyes, remove the contact lenses, stop using MeniCare Soft solution, and consult your eye care professional or practitioner.</p> <p>If any serious incidents while using the care product occur, report it to the authorised representative and/or the manufacturer, and the competent authority.</p>
CONTENTS
<p>MeniCare Soft is an isotonic, aqueous solution containing polyhexanide 0.0001% as the active ingredient and preservative, macroglycyerol hydroxystearate 60, disodium edetate and isotonic agents.</p>
PACKAGING
<p>MeniCare Soft is available in 70 mL and 360mL bottles, including disposable lens case.</p>

People allergic to any ingredient with MeniCare Soft Solution should not use this product. Never use MeniCare Soft Solution for heat disinfection.

**ADVERSE REACTIONS**  
The following problems may occur: eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on the eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eye, unusual eye secretions, redness of the eye, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), or dry eyes. If you experience any of the above in the eyes, remove the contact lenses, stop using MeniCare Soft solution, and consult your eye care professional or practitioner.

If any serious incidents while using the care product occur, report it to the authorised representative and/or the manufacturer, and the competent authority.

CONTENTS
<p>MeniCare Soft is an isotonic, aqueous solution containing polyhexanide 0.0001% as the active ingredient and preservative, macroglycyerol hydroxystearate 60, disodium edetate and isotonic agents.</p>
PACKAGING
<p>MeniCare Soft is available in 70 mL and 360mL bottles, including disposable lens case.</p>

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Lease read carefully and conserve for future riferimento.

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