## Instructions for Use: tolofocon A or tsiifoon A for Menicon Rigid Gas Permeable (RGP) contact lens

<b>Important</b>
<ul style="list-style-type: none"><li>Read the following instructions carefully before use and keep this document in a safe place for future reference.</li> <li>Always follow the instructions in this document and those of your eye care professional or practitioner for appropriate use of contact lenses and lens care products including lens cases.</li></ul>

### INTRODUCTION

Menicon RGP contact lenses are made of oxygen permeable materials, i.e., tolofocon A (Menicon EX) and tsiifoon A (Menicon Z). The Menicon RGP contact lenses are individually packaged in a plastic container and immersed in PHMB solution.

### INDICATIONS

Menicon RGP contact lenses are intended for correction of refractive ametropia (myopia, hyperopia, presbyopia, and/or astigmatism) in aphakic or non-aphakic disease-free eyes.

- Menicon EX is intended for daily use up to 17 days extended wear.
- Menicon Z, Menicon Z-a, Menicon EX-Z, and Menifocal Z are intended for daily up to 30 days extended wear.
- Menifocal Z is indicated for presbyopia.
- Menicon RGP contact lenses other than contact lenses with fresh lenses within 2 years. Be sure to consult your eye care professional or practitioner for the lens wearing schedule based on your individual needs. Only your eye care professional or practitioner can determine if you can use extended wear contact lenses. Instruct the wearing schedule and maintenance procedure. Always dispose of the contact lenses after the instructed wearing period, and wear the contact lenses beyond the instructed wearing schedule and period increases the risk of eye problems.

### WEARING RESTRICTIONS

Clean and disinfect the contact lenses after lens removal using the cleaning and disinfecting regimen recommended by Menicon (MeniCare Pure or MeniCare Plus for multipurpose solution and Menicon Progent for intensive protein remover) or by your eye care professional or practitioner for details on lens care products, see “CARE FOR THE CONTACT LENSES”. Use eye care professional or practitioner for the lens wearing schedule based on your individual needs. Only your eye care professional or practitioner can determine if you can use extended wear contact lenses. Instruct the wearing schedule and maintenance procedure. Always dispose of the contact lenses after the instructed wearing period, and wear the contact lenses beyond the instructed wearing schedule and period increases the risk of eye problems.

### CONTRAINDICATIONS

- Do not wear the contact lenses if you have any of the following conditions.
  - Acute and subacute inflammation or infection of the anterior chamber of the eyes
  - Any disease, injury, or abnormality that affects the cornea, the conjunctiva or eyelids
  - Corneal hypoxia/edema (reduced corneal oxygenation)
  - Any systemic disease that may adversely affect the eyes by wearing contact lenses
  - Allergic reactions that may be induced or exaggerated by wearing contact lenses and/or using lens care solutions
  - Any active corneal infection
  - Red or irritated eyes
  - Incomplete corneal healing following eye surgery

### WARNINGS

- Because of the following warnings pertaining to contact lens wear.
  - Eye problems arising from wearing contact lenses and use of lens care products, such as corneal ulcers, can develop rapidly and lead to loss of vision. If you experience any discomfort in the eyes, excessive tearing, vision changes, redness in the eyes or other problems, immediately remove the lenses and consult your eye care professional or practitioner.

### PRECAUTIONS

- Handle the contact lenses with care (be careful not to drop them).
- Do not touch the contact lenses with your fingers.
- Put in the contact lenses before applying cosmetics, but do not apply cosmetics, lotions, creams, or sprays while handling the contact lenses.
- Consult your eye care professional or practitioner before using any medicine in your eyes.
- If the contact lenses feel dry on your eyes, apply a few drops of the lubricating wetting solution (recommended by your eye care professional or practitioner) directly to the eyes and wait until the lenses can move freely on the eyes before removing them. If the discomfort still continues, immediately consult your eye care professional or practitioner.
- Do not use any eye drops or eye medicines and fumes while wearing the contact lenses.
- If Menicon Progent (intensive protein remover) or any chemicals are splashed into the eyes, immediately flush the eyes with tap water, remove the contact lenses, and consult your eye care professional or practitioner, or visit a hospital emergency room.
- Consult your eye care professional or practitioner for instructions on wearing the contact lenses during any strenuous sports or water activities, or for specific occupations.
- Follow-up visits are necessary to assure the continued health of your eyes.
- Consult your eye care professional or practitioner if you wish to change the lens type or parameters.
- Do not use contact lenses that you wear at room temperature.
- Store the contact lenses at room temperature.
- Consult your eye care professional or practitioner if you need to store the contact lenses for extended periods.
- Consult your eye care professional or practitioner before wearing the contact lenses if you have not worn the lenses for extended periods.
- If the contact lenses become dry in a lens case, consult your eye care professional or practitioner.
- Do not share the contact lenses with other people.

### ADVERSE REACTIONS

If any of the following problems occur, immediately remove the contact lenses and examine them to be sure that they are free of any damage. If the contact lenses are not damaged, clean and disinfect the lenses before wearing them again. If the problem still continues, remove the lenses and seek prompt treatment. Serious eye damage may result such as infection, corneal ulcers, neovascularization, and iritis.

- Eye stinging, burning, itching, or any other pain in the eyes
- Constant foreign body sensation or a few drops of the lubricating wetting solution
- Excessive tearing, unusual eye secretions, redness, reduced visual acuity, blurred vision, halos, photophobia, or dry eyes
- If any serious incidents while using the contact lenses occur, report it to the authorized representative and/or the manufacturer, and the competent authority.

### HANDLING THE CONTACT LENSES

- Always wash, rinse, and dry your hands thoroughly before handling the contact lenses.
- Do not wear the contact lenses if the plastic containers have been opened or damaged, or if the expiration date has been passed.
- Clean and rinse the contact lenses before first use using MeniCare Pure, MeniCare Plus, or lens care products recommended by your eye care professional or practitioner for details on lens care products, see “CARE FOR THE CONTACT LENSES”.
- Always handle the left and right contact lenses in the same sequence to avoid mixing up the lenses.

### Putting in contact lenses:

The general procedure for putting in contact lenses is described below. Consult your eye care professional or practitioner for more details as necessary.

- Unscrew the stopper of the lens case and rinse the contact lens in the lens holder.
- Pick up the lens from the lens holder and examine it to be sure that it is free of damage or foreign bodies.
- Place the lens on the tip of the forefinger of your dominant hand with the concave side facing up.
- Hold your eyelids wide apart by pulling the upper lid with the forefinger of your non-dominant hand and the lower lid with the middle finger of your pulling the upper lid.
- With your left hand, look up to your eye while looking straight ahead, and gently put the lens on the cornea.
  - Do not press the lens against your eye.
  - Look down, release the upper lid first then the lower lid, and blink gently.
- Repeat this procedure for the other lens.
- If you feel discomfort after putting in the contact lens, this may be due to the presence of foreign body. Remove and rinse the lens, and then put in the lens again. If the discomfort still continues, remove the lens and consult your eye care professional or practitioner.
- If your vision is blurred after putting in the contact lenses:
  - If the contact lenses are not centered on the cornea:
    - Look into the mirror, and gently manipulate the edge of the lens to move it onto the cornea while opening your eye. Use finger pressure on the edge of the upper or lower lid while the lens is centered on the cornea. Alternatively, remove and clean the lens, and then put in the lens again.
    - If the contact lenses are centered on the cornea:
      - Remove the lenses. Possible causes are as follows.
        - The lens surface is too dry.
        - The lenses are not wearing evenly.
      - If the lens surface is still blurred after clearing the problems mentioned above, remove the contact lenses and consult your eye care professional or practitioner.
      - If you feel significant discomfort while wearing the contact lenses, immediately remove the lenses and consult your eye care professional or practitioner.

### Removing contact lenses:

- Open your eyes wide and look straight ahead.
- Sharply pull the outside corner of your eye outward and catch the contact lens in the palm of your other hand.
- Repeat this procedure for the other lens.

### Method 2:

- Place the forefinger at the base of the lower eyelashes and the other forefinger on the upper eyelid.
- Open your eyes wide and move the forefingers towards each other, exerting a slight pressure to separate the contact lens from the eyes.
- Repeat this procedure for the other lens.

### CARE FOR THE CONTACT LENSES

- Always use lens care products recommended by Menicon (MeniCare Pure or MeniCare Plus for multipurpose solution and Menicon Progent for intensive protein remover) or by your eye care professional or practitioner. Never use care products for soft contact lenses.
- Do not use tap water for the care of the contact lenses and lens cases. Use of tap water increases the risk of microbial infection.
- Always use fresh unexpired lens care products.
- Do not alternate or mix lens care products unless indicated on the product instructions or recommended by your eye care professional or practitioner.
- Heat disinfection is not recommended.
- Store the contact lens completely immersed in storage solution when not in use.
- Clean and rinse the lens case with care and allow it to air dry after each use. Replace lens cases at regular intervals.

### Maintenance after daily wear

- Clean and rinse the contact lens after lens removal using MeniCare Pure, MeniCare Plus, or lens care products recommended by your eye care professional or practitioner, put the lenses in lens holders, and keep the lenses in a lens case. Use Menicon Progent or an intensive protein remover recommended by your eye care professional or practitioner once a week. For details, read “Instructions for Use” for the lens care products.

### Maintenance after extended wear

- Clean and rinse the contact lens after lens removal using MeniCare Pure, MeniCare Plus, or lens care products recommended by your eye care professional or practitioner, and then use Menicon Progent or an intensive protein remover recommended by your eye care professional. For details, read “Instructions for Use” for the lens care products.

### SYMBOLS USED ON THE PACKAGE

SYMBOL				
DEFINITION	Consult Instructions for Use	Barcode Code	Use Your Eyes	Non-sterile
SYMBOL				
DEFINITION	MeniCare Solution	Unique Identification	Eye Hygiene	Do not use with lens care solution or contact lens
SYMBOL				
DEFINITION	Diameter	Base Curve	Depth	Center Thickness
SYMBOL				
DEFINITION	Add power	Add power	Add power	Add power

Manufacturer: Menicon Co., Ltd. 3-21-19, Aoi, Naka-ku, Nagoya, 460-0006 Japan www.menicon.com
Authorized Representative: Menicon Holdings B.V., Waanderweg 6, 7812 HZ, Emmen, The Netherlands
Date of Issue: 2024-08
©2024, Menicon Co., Ltd.

## Brugsanvisning: tolofocon A eller tsiifoon A til formstabile (RGP) Menicon-kontaktlinser

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