CO D.

227025

Rev.

01

Data

2024/06

LINEA

Soleko

Prodotto

SAO 500 ML

Tipo

label

FUSTELLA

195X127 mm

COLOR

CMYK

Cool Gray 11 C

EN PROPERTIES AND INDICATIONS: **Saline solution is suitable for rinsing all types of contact lenses. pH balanced formula similar to that of human tears, mild on the eye.** INSTRUCTIONS FOR USE: Wash, rinse and dry your hands before handling the lenses. Saline can be used to rinse contact lenses after cleaning, just before disinfection with disinfecting solutions. After disinfection, before application, the lenses can also be rinsed with Saline as an alternative to multipurpose solutions. Close the bottle after use. **PRECAUTIONS:** Discard contents of bottle 90 days after opening. Do not use if tamper-evident seal is damaged. Store between 5 and 30°C. Do not use the product after the expiry date. **WARNINGS:** In the event of persistent eye irritation, discontinue use immediately, remove the lens from the eye and consult your contact lens practitioner. Keep all contact lens care products away from children. Do not allow the nozzle tip to come into contact with any surfaces. Do not rinse the contact lenses or lens case directly with tap water. Never swallow the Saline solution. Any serious incidents that occur in relation to the device should be reported to Soleko S.p.A. and the competent authority of the Member State in which the user is established. **CONTRAINDICATIONS:** Do not use the product if you are allergic or sensitive to any of the ingredients. **CONTENTS:** Sodium Chloride 0.005%, Sodium Chloride, Dibasic Sodium Phosphate, Monobasic Sodium Phosphate, Disodium EDTA, Purified water.

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CH REF Menicon Swiss GmbH, Dornacherstrasse 27, 4600 Olten, Switzerland

Please Recycle!

SOLEKO S.p.A. - Via Ravano snc 03037 - Pontecorvo (FR) - ITALIA

STERILE

023384 167307

GENTLE, PH-BALANCED
SALINE SOLUTION

**SALINE
SOLUTION**

Suitable for rinsing all types of
contact lenses and lens cases

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